








Application information Quadrant Universal LC Posterior restoration


- 


1. 46 Distal cavity
- 


2. Prepare the cavity according to the general directions for composite restorations.
- 


3. Clean and dry the preparation. The use of rubberdam is recommended.
- 


4. Apply Quadrant Total Etch onto enamel and dentin and leave for 15-30 seconds. Rinse thoroughly for at least 5 seconds. Remove excess moisture with a gentle stream of air.
- 

5. Apply 3 layers of Quadrant Uni-1-Bond. Wait for 15 seconds and lightcure for 20 seconds.
- 

6. Apply Quadrant Universal LC. If necessary in several layers of max. 2 mm thickness.
- 

7. Contour the surface of the last layer.
- 

8. Light cure every layer for 20 seconds
- 

9. Finish and polish with fine diamond, flexible discs and silicon polishers.
- 

10. Result after restoration.